

RapidPort® EZ Port Applier

DIRECTIONS FOR USE (DFU)

Rx Only



RapidPort® EZ Port Applier

Ref. No. C-20390 RapidPort® EZ Port Applier

INTRODUCTION

The RapidPort® EZ Port Applier is an optional accessory for the LAP-BAND AP® Adjustable Gastric Banding System and is designed to be used only with LAP-BAND AP® systems that include the RapidPort® EZ Access Port. It is designed as an alternative method of access port fixation, taking the place of suturing or other fixation method.

The RapidPort® EZ Port Applier enables the surgeon to anchor the RapidPort® EZ Access Port to the patient's abdominal muscle fascia in less time than manually suturing an access port. Qualified LAP-BAND® System surgeons should be familiar with the LAP-BAND® system procedure prior to the use of the RapidPort® EZ Port Applier.

DESCRIPTION

The RapidPort® EZ Port Applier Kit contains:

- RapidPort® EZ Port Applier Tool, one each

INDICATIONS FOR USE

The RapidPort® EZ Port Applier is indicated for use for initial implantation or replacement of the RapidPort® EZ Access Port.

CONTRAINDICATIONS

In addition to the following contraindications, the RapidPort® EZ Port Applier has the same contraindications that are listed in the LAP-BAND AP® System Directions for Use (DFU):

1. The RapidPort® EZ Port Applier is contraindicated if the surgeon is unable to place, for any reason, the access port flat against the abdominal muscle fascia.
2. The RapidPort® EZ Port Applier is contraindicated if fat on the abdominal anterior muscle sheath fascia cannot be adequately cleared for proper fixation of the access port.
3. The RapidPort® EZ Port Applier is contraindicated for patients who are known to have, or are suspected to have an allergic reaction to materials contained in the system or who have exhibited a pain intolerance to implanted devices.

WARNINGS AND PRECAUTIONS

Patients should be advised that the LAP-BAND® system is a long-term implant. Medical management of adverse reactions may include explantation. Revision surgery for explantation and replacement may also be indicated to achieve patient satisfaction.

The stainless steel anchors on the Access Port are sharp and may penetrate surgical gloves if not handled appropriately. Care must be taken by the surgical team while handling and attaching the Access Port onto the RapidPort® EZ Port Applier. Always keep the black safety cover on the access port until the access port is loaded onto the RapidPort® EZ Port Applier and the access port is about to be placed in the patient.

The access port's black safety cover is not to be implanted.

The access port must be securely fastened to the patient's rectus fascia with all four of the stainless steel anchors firmly embedded in the patient's fascia. If this is not achieved, the access port may become loose and inaccessible for post-operative adjustments, thus requiring revision surgery.

The RapidPort® EZ Port Applier is not intended to be re-sterilized or re-used. Cleaning and autoclaving processes may cause damage to components resulting in improper function and possible failure of the RapidPort® EZ Port Applier. Reuse of the device can cause infection to the patient.

The RapidPort® EZ Access Port and Applier components contain nickel, which is a known allergen.

ADVERSE EVENTS

Adverse events that may result from the use of this product include:

1. The risks associated with the medications and methods utilized in the surgical procedure.
2. The risks associated with any surgical procedure and the patient's degree of intolerance to any foreign object implanted in the body.
3. Infection can occur in the immediate post-operative period or years after insertion of the device. In the presence of infection or contamination, removal of the device is indicated.

Please refer to the LAP-BAND AP® with RapidPort® EZ system DFU for additional adverse event information.

HOW SUPPLIED

The RapidPort® EZ Port Applier is provided sterile for single use only.

The RapidPort® EZ Port Applier should be stored in a clean, dry location (standard hospital supply storage).

INSTRUCTIONS FOR USE

ACCESS PORT PLACEMENT

1. Prime the access port as described in the LAP-BAND AP® with RapidPort® EZ system DFU or RapidPort® EZ Access Port Kit DFU.
2. After the band is placed, the band tubing is brought outside the abdomen. The tubing may be shortened to tailor the position of the access port to the patient while avoiding tension between the port and the band. Remove the end plug from the band tubing. Then, push the access port tubing with stainless steel connector onto the band tubing until it is flush against the band tubing (**Figure 1**).

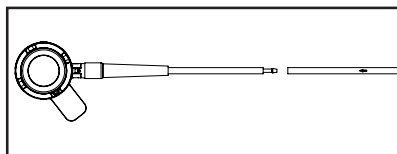


Figure 1. Band to port tubing connection

3. Hold the applier as shown (**Figure 2**). Ensure that the RapidPort® EZ Port Applier trigger is in the fully opened position (**Figure 3**).

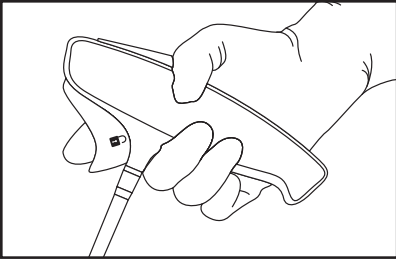


Figure 2. Proper hand positioning when holding the applier

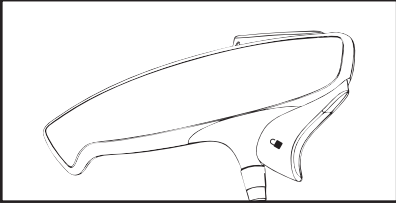


Figure 3. Port Applier trigger in the open position

4. Insert the access port into the RapidPort® EZ Port Applier deployment head. Ensure the port tubing is parallel to the Applier handle. Tubing can be pointing in the direction of the handle or in the opposite direction (**Figure 4**). The port will snap into place.

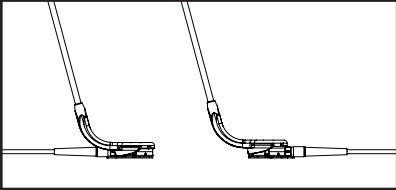


Figure 4. Loading the access port into the port applier deployment head

5. Pull the black safety cover off the bottom of the access port. Discard the safety cover.

CAUTION: Do not squeeze or attempt to depress the handle to the RapidPort® EZ Port Applier. Doing so will deploy the fasteners prematurely.

CAUTION: Failure to create a stable, smooth path for the access port tubing, without sharp turns or bends, can result in tubing breaks and leakage. In order to avoid incorrect placement, the access port should be placed lateral to the trocar opening and a pocket must be created for the access port, so that it is placed far enough from the trocar path to avoid abrupt kinking of the tubing. The tubing path should point in the direction of the access port connector so that the tubing will form a straight line with a gentle arching transition into the abdomen. The tubing should be perpendicular to the midline of the patient. (**Figure 5**)

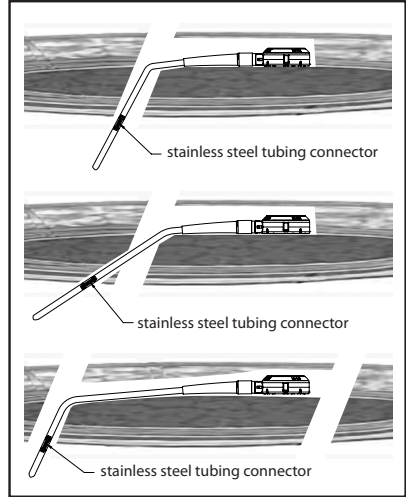


Figure 5. Port placement options

6. Verify that the fat has been fully cleared and fascia is visible in the selected area. Ensure an adequate incision size to accept the RapidPort® EZ deployment head. Introduction of the deployment head from an angled position may facilitate insertion. Pocket dissection must be adequate to allow for a smooth transition of the tapered connector into the abdomen. Incision length can be minimized by temporarily removing the retractors until the instrument's deployment head is positioned correctly in the abdomen. Position the RapidPort® EZ deployment head flat against the patient's rectus muscle fascia to ensure that all four stainless steel fasteners will fully engage the fascia and/or muscle tissue.

Note: Tubing must be placed in the abdomen to ensure the access port is flat. Exposed tubing may cause a tilting of the access port and prevent the ability of all fasteners to engage in the fascia.

CAUTION: When using retractors, take care not to apply undue force to the RapidPort® EZ deployment head.

7. Once the deployment head is properly positioned in the dissected pocket, apply pressure to the deployment head by pushing downward on the applier handle to ensure the access port is flat against the fascia and the instrument is steadied for firing. Firmly squeeze the RapidPort® EZ Port Applier trigger until it is depressed (**Figure 6**) and the stainless steel anchors are deployed into the underlying fascia. *Do not release the trigger.*

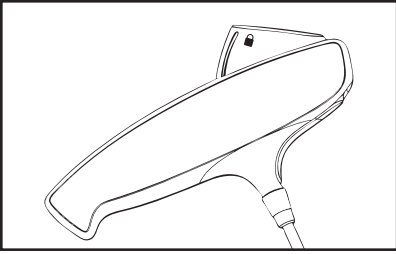


Figure 6. Port applier trigger depressed for deployment of port anchors

8. To disengage the port applier, fully depress the trigger and hold it while gently lifting the deployment head off the port and back a few millimeters. Once the head has cleared the port, you may remove your hand from the trigger. Finally, remove the applier from the incision site. Angling the instrument, once it is detached from the access port, can aid in the removal of the instrument.
9. Ensure that the access port's stainless steel anchors are fully engaged into fascia by running a finger around the base of the access port.

CAUTION: In order for the access port to be properly affixed, all four stainless steel fasteners must be well within the patient's fascia and/or underlying muscle. Otherwise, the access port can become detached and subsequent removal and/or replacement may become necessary.

UNLOCKING ACCESS PORT AND REPOSITIONING

To ensure durable fixation of the access port, the stainless steel anchors are locked into place. In the event the access port needs to be repositioned, re-engage the access port with the tool in the locked position. Although your fingers may rest on the trigger, do not depress the trigger at any time. Once the head is over the port, it can engage the port. Once engaged, press the trigger on the top of the instrument (button has lock symbol). Fully depress the trigger to completely retract the anchors.

RETURNED GOODS POLICY

Authorization must be received from your Apollo Endosurgery Account Manager prior to return of the merchandise. Merchandise returned must have all the manufacturer's seals intact to be eligible for credit or replacement. Returned products are subject to restocking charges. No credit will be issued on marked or damaged boxes with stickers.

LIMITED WARRANTY, LIMITATION OF LIABILITY AND DISCLAIMER OF OTHER WARRANTIES

There is no express or implied warranty, including without limitation any implied warranty of merchantability or fitness for a particular purpose, on the Apollo Endosurgery, Inc. product(s) described in this publication. To the fullest extent permitted by applicable law, Apollo Endosurgery, Inc. disclaims

all liability for any indirect, special, incidental, or consequential damages, regardless of whether

such liability is based on contract, tort, negligence, strict liability, products liability or otherwise. The sole and entire maximum liability of Apollo Endosurgery, Inc., for any reason, and buyer's sole and exclusive remedy for any cause whatsoever, shall be limited to the amount paid by the customer for the particular items

purchased. No person has the authority to bind Apollo Endosurgery, Inc. to any representation or warranty except as specifically set forth herein. Descriptions or specifications in Apollo Endosurgery, Inc. printed matter, including this publication, are meant solely to generally describe the product at the time of manufacture and do not constitute any express warranties or recommendations for use of the product in specific circumstances. Apollo Endosurgery, Inc. expressly disclaims any and all liability, including all liability for any direct, indirect, special, incidental, or consequential damages, resulting from reuse of the product.

AUTHORIZED TRAINING PROGRAM AND PRODUCT INFORMATION

LAP-BAND® System placement is an advanced laparoscopic procedure. Surgeons planning LAP-BAND® System placement must participate in a LAP-BAND® System training program authorized by Apollo Endosurgery or an authorized Apollo Endosurgery distributor. This required training program is specific to the Apollo Endosurgery LAP-BAND® System and does not qualify for use with other gastric bands.









The RapidPort® EZ Port Applier contains no latex or natural rubber materials.

Patented. See: www.apolloendo.com/patents

For additional information please contact:

Manufacturer

Apollo Endosurgery, Inc.
Austin, Texas 78746 U.S.A.
Tel: (512) 279-5100
Fax: (512) 279-5105

	<p>Sterilized Using Irridation</p>
	<p>Caution. See Instructions For Use</p>
	<p>Do Not Use If Package Is Damaged</p>
	<p>Reference Number</p>
	<p>Product Does Not Contain Latex</p>
	<p>Lot Number</p>
<p>Rx Only</p>	<p>CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician</p>
 <p>YYYY-MM-DD</p>	<p>Use By Year, Month</p>
	<p>Single Use Only. Do Not Reuse.</p>



Manufacturer
Apollo Endosurgery, Inc.
Austin, TX 78746 USA

Assembled in U.S.A

Tel: (512) 279-5100
Fax: (512) 279-5105

www.apolloendo.com

L3645 Rev.01 [3645-01]

01/2015

GRF-00256-00R01